




Prevent Pregnancy Naturally

LADY-COMP®

... the safe
fertility computer
for over 25 years

NOW available
in the USA

LADY-COMP®

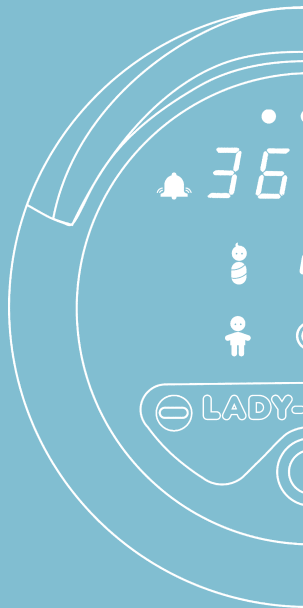

What?

Based on your morning temperature taken orally, LADY-COMP® determines your fertile days by recording, analyzing and storing your menstrual cycle data.

It monitors your daily fertility status and will advise you of the days when intercourse may lead to pregnancy: on the day you ovulate – and five days before ovulation.

LADY-COMP® is a reliable, natural, hormone-free fertility awareness method which can be easily used at home. This reliable fertility computer comes complete with an integrated rechargeable battery and its own power supply. Perfect for everyday use.

Why?

On average a woman can become pregnant on six days per cycle – five days before ovulation and on the day of ovulation. In order to avoid pregnancy in this short period of time, many women accept day in, day out, considerable compromises to their health, putting a heavy strain on their health or love life.

Does it have to be that way? We say: no!

99.3 percent effective

LADY-COMP® is 99.3% effective right from the start. It has a clinical proven Pearl-Index of 0.7 and 25 years of supporting history.

How ?

Wake up – Measure – Done!

Every morning you take your wake-up temperature by placing LADY-COMP®'s temperature sensor under your tongue for 50 - 60 seconds. LADY-COMP® will automatically request information regarding menstrual data on relevant days of the cycle. It's that simple!

LADY-COMP® reliable gives a green or red light for the next 24 hours.

- green means “infertile”
- red means “fertile”
- yellow means that the computer is still in its learning phase

Value

- One time investment
- NO follow-up costs
- Over 10 years life expectancy
- High Quality
- Life-long service support

LADY-COMP® can be upgraded with a special planning software to maximize your chances of becoming pregnant.

For more information please visit us at www.lady-comp.com



VALLEY
ELECTRONICS

Valley Electronics LLC
25505 Collins Wharf Rd.
Eden, MD 21822
USA

Tel 1.877.925.LADY (5239)
Fax 1.877.925.5239

www.lady-comp.com
info@lady-comp.com